

# 3M™ Medical Specialties

## Technical Information Bulletin

December 13, 2007

### **Recommended Cleaning of 3M Press-On™ Optics**

Under certain storage conditions a haze may form on the surface of the Press-On™ Optic. Prior to application both Press-On™ optic and spectacle lens should be free of any soil or other contaminants.

We recommend using a 3M Scotch Brite™ lens cleaning cloth. The unique 3M micro-ribbon fiber technology lifts dust, moisture and oils from the grooved Fresnel pattern without the use of chemicals and won't scratch the surface.

### **Recommended Cleaning Method 3M Scotch Brite™ Cleaning Cloth**

When using the 3M Scotch Brite™ cleaning cloth, gently wipe both surfaces of the Press-On™ optic (in the direction of the Fresnel pattern) until surface is clean and streak-free.

### **Alternate Cleaning Method Liquid Dish Detergent**

A weak solution of liquid dish detergent can also be used to clean the Press-On™ optic. Follow the cleaning with a thorough rinsing of the lens in warm water and pat or blot dry with a soft, nonlinting cloth.

3M Medical Specialties  
3M Center, Building 275-5W-05  
St. Paul, MN 55144-1000  
USA  
800-228-3957  
[www.3M.com/medicalspecialties](http://www.3M.com/medicalspecialties)